

WHAT COLOR TUBE TO DRAW

ALL TESTS LISTED CAN BE TRANSPORTED TO LAB VIA ROUTINE COURIER

***** Tubes with gel in them should be spun prior to transit *****

ALBUMIN
ALKALINE PHOSPHATASE
AMYLASE
ANTINUCLEAR ANTIBODY (ANA)
BASIC METABOLIC PANEL (BMP)
BETA (QUANTITATIVE) HCG
BILIRUBIN
BLOOD TYPE & ANTIBODY SCREEN- TALL LAV
BLOOD UREA NITROGEN (BUN)
B-TYPE NATRIURETIC PEPTIDE (BNP) -TALL LAV
CALCIUM
CARBAMAZEPINE (TEGRETOL)
CARBON DIOXIDE
CBC OR CBC WITH DIFF
CHLORIDE
COMPREHENSIVE METABOLIC PANEL (CMP)
CORTISOL
CRP-C-REACTIVE PROTEIN
CREATININE
CREATININE KINASE
CYCLOSPORINE
D-DIMER
DIGOXIN
ELECTROLYTES
FERRITIN
FOLATE
FREE T3
FREE T4
GAMMA GLUTAMYLTRANSFERASE(GGT)
GENTAMYCIN
GLUCOSE
GOT
GPT
HCG (QUALITATIVE)
HEMOGLOBIN A1C
HEPATIC FUNCTION PANEL
HEPATITIS PANEL- DRAW GOLD & TALL LAV
HIV- DRAW GOLD & TALL LAV

IRON / IRON BINDING STUDIES, TIBC, % SAT
LEAD
LIPASE
LIPID PANEL
LITHIUM
MAGNESIUM
METHYLMALONIC ACID
NICOTINE AND METABOLITES - 2 TUBES
PARATHYROID, INTACT - TALL LAV
PHENYTOIN (DILANTIN)
PHOSPHORUS
POTASSIUM
PROTEIN
PSA
PT-INR
PTT <small>Deliver to the lab within 4 hours of draw</small>
RETICULOCYTE COUNT
RHEUMATOID FACTOR
RUBELLA IGG
SED RATE- TALL LAV
SERUM PROTEIN ELECTROPHORESIS
SIROLIMUS (RAPAMUNE)
SODIUM
SYPHILIS ANTIBODY
TACROLIMUS (PROGRAF)
TESTOSTERONE
THYROGLOBULIN ANTIBODY
THYROGLOBULIN LEVEL
THYROID PEROXIDASE ANTIBODY
T3 TOTAL
T4 TOTAL
TROPONIN
TSH
URIC ACID
VALPROIC ACID (DEPAKOTE)
VANCOMYCIN
VITAMIN B12
VITAMIN D 1, 25-DIHYDROXY
VITAMIN D 25-HYDROXY

DRAW TUBES IN THE FOLLOWING ORDER

1. BLOOD CULTURE	7. LITH HEP (TALL LIGHT GRN)
2. BLUE TOP	8. TALL LAV (1 TUBE PER TEST)
3. RED TOP	9. SHORT LAVENDER
4. NO ADDITIVE (DARK BLUE)	10. EDTA (DARK BLUE)
5. SST/GOLD TOP	11. ACD-A OR ACD-B (FAT YELLOW)
6. SODIUM HEP (SHORT GRN)	12. QUANTIFERON TB KIT